
PRODUCT DATA SHEET Disposable Face Mask, Ear - Loop 3 ply

Face Mask, 3ply-FMEL3

Description

Face Mask, 3 ply is made of Polypropylen Spunbond and Meltblown nonwoven fabrics with adjustable nose strip and flat or round elastic ear - loop to give basic protection to the user. It's mainly used in medical, dental, clenroom, critical environment, food handling/production and industrial manufaturing.



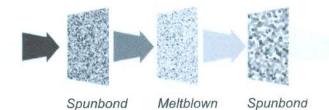

Components

1st layer Polypropylene Spunbond nonwoven

2nd layer Meltblown Nonwoven

3rd layer Polypropylene Spunbond nonwoven

Nose strip PE wire (Metal Free)

Ear - Loop Flat elastic or round elastic (latex free)

Specifications

Width across face 175 mm +/-2 mm

Depth pleated 95 mm +/-2 mm

Depth in full 175 mm + /-2 mm

Nose strip 95 mm +/-2 mm

Loop size $155 \,\mathrm{mm} + / - 5 \,\mathrm{mm}$

Packing 50 pcs per dispenser box,

20 dispenser boxes per carton (1000 pcs)

Available White, Blue, Green colour, as requested

With cartoon pattern up to three colours

Inner and outer ear loop with any colour as requested

Benefit and standards

- BFE (Bacterial Filtration Efficiency) 99% (tested by Nelson lab)
- Excellent breathability
- Non irritating, fiber glass free fabric
- Non woven spunbond 100% Polyplopylene
- Hypoallergenic
- Splash resistant
- Comply with EN 14683, CE marking, ISO 9001 and 14001
- Ultrasonic welding manufacturing process
- Hygienic





Sponsor: Natachnon Dhamatoh Med-Con (Thailand) Co., Ltd. 20 Soi Ruammit, Rama 6 Road Samsennai, Phayathai Bangkok TH10400 THAILAND

EN 14683:2005 Bacterial Filtration Efficiency and Differential Pressure Final Report

Test Article: #MJO12518B

Laboratory Number: 635599

Study Received Date: 21 May 2012

Test Procedure(s): Standard Test Protocol (STP) Number: STP0004 Rev 06

Summary: The bacterial filtration efficiency (BFE) test is performed to determine the filtration efficiency by comparing the bacterial control counts to test article effluent counts. A suspension of Staphylococcus aureus was aerosolized using a nebulizer and delivered to the test article at a constant flow rate. The aerosol droplets were drawn through a six-stage, viable particle, Andersen sampler for collection. This procedure allows a reproducible bacterial challenge to be delivered to test materials. The differential pressure (ΔP or Delta P) test determines the breathability by measuring the differential air pressure on either side of the test article using a manometer, at a constant flow rate. Testing was conducted as directed in Annex B (BFE testing) and Annex C (Delta P testing) of EN 14683:2005. All test method acceptance criteria were met.

Test Side:

Outside Surface

BFE Area Tested:

~3.0 inches (75 mm) diameter

BFE Flow Rate: 28.3 L/min (1 cubic foot per minute(CFM))

Delta P Flow Rate: 8 L/min

Conditioning Parameters: 65 ± 2% relative humidity and 20 ± 2°C

Results: Test articles with a filtration efficiency greater than or equal to 95% meet the performance requirements of EN 14683:2005 as Type I and/or Type IR. Test articles with a filtration efficiency greater than or equal to 98% meet the performance requirements of EN 14683:2005 as Type II and/or Type IIR.

Test articles with a differential pressure less than 29.4 pascals (Pa) meet the performance requirements of EN 14683:2005 as Type I and/or Type II. Test articles with a differential pressure less than 49.0 Pa meet the performance requirements of EN 14683:2005 as Type IR and/or Type IIR.

Unit Number	Percent BFE (%)	ΔP (Pa/cm²)
1	99.8	23.1
2	99.8	26.3
3	99.8	24.0
4	>99.9	23.6
5	>99.9ª	23.5

^a There were no detected colonies on any of the Andersen sampler plates for this test article.

Mean Positive Control Count:

2,561 colony forming units (CFU)

Negative Control Count:

<1 CFU

Mean Particle Size (MPS): Test Article Dimensions: 2.9 µm 171 mm x 152 mm

Sarah Smit, B.S.

29 May 3012 Study Completion Date

FRT0004-0002 Rev 6 Page 1 of 1